



The current Prolex™ line-up



Pro-Lab Diagnostics U.K.
7 Westwood Court
Neston, South Wirral, Cheshire
CH64 3UJ
United Kingdom
Tel: 0151 353 1613
Fax: 0151 353 1614
E-Mail: uksupport@pro-lab.com

Pro-Lab Diagnostics Canada
20 Mural Street, Unit #4
Richmond Hill, ON, L4B 1K3
Canada
Toll Free: 1-800-268-2341
Tel: (905) 731-0300
Fax: (905) 731-0206
E-Mail: support@pro-lab.com

Pro-Lab Diagnostics U.S.A.
9701 Dessau Road, Suite #802
Austin, TX, 78754-3941
U.S.A.
Toll Free: 1-800-522-7740
Tel: (512) 832-9145
Fax: 1-800-332-0450
E-Mail: ussupport@pro-lab.com




www.pro-lab.com

Prolex™ Staph Xtra Latex Kit

PRO-LAB DIAGNOSTICS proudly presents the new Prolex™ line of blue latex agglutination systems for use in the rapid identification of clinically relevant pathogens.

The Prolex™ blue line of products employs a precise, industry leading manufacturing process, achieved after many years of research and development. Coupled with a first class QC/QA commitment, Prolex™ stands as the fastest, most accurate and reliable product line on the market today.

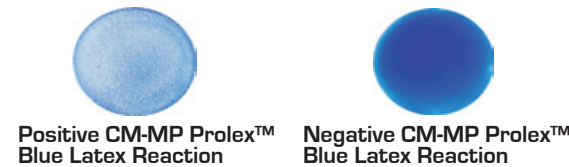

New Agglutination Card Design

The Prolex™ Staph Xtra kit utilizes CM-MP particles which have been sensitized with fibrinogen and IgG that are specific for capsular types 5 and 8 of *S.aureus*. When staphylococcal colonies which possess at least one of clumping factor, protein A and / or capsular 5 or 8 antigens are mixed with the latex reagent, the blue latex particles agglutinate strongly within 20 seconds against a white background.

PRODUCT FEATURES

- As few as two colonies required from primary culture
- Enhanced results with blue carboxylate-modified microparticles (CM-MP)
- Detects Coagulase, Protein A, capsular 5 and 8 antigens
- Definitive results obtained within 20 seconds
- Contains negative control
- Available in 2 convenient sizes, 100 tests and 300 tests
- Includes new agglutination card design

PRODUCT AVAILABILITY

PL.1080

Staph Xtra Latex Kit (100 Tests)

Contains:

- Staph Xtra Latex Reagent 2 x 2.5ml
- Negative Control Latex Rgt 5.0ml
- Agglutination Cards
- Mixing Sticks

PL.1081

Staph Xtra Latex Kit (300 Tests)

Contains:

- Staph Xtra Latex Reagent 2 x 7.5ml
- Negative Control Latex Rgt 2 x 7.5ml
- Agglutination Cards
- Mixing Sticks

PL.1086

Negative Control 7.5ml

PL.091

Mixing Sticks

PL.092-48

Agglutination Cards